**EN:** Filtering half masks to protect against particles

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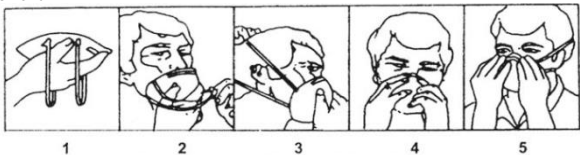
**CE 0086 EN149:2001+A1:2009**

**NR:** That masks marked NR shall not be used for more than one shift.

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## EN: INSTRUCTIONS FOR USE

**IMPORTANT:** Single-use respirators against solid and liquid aerosols are classified according to one of three categories - FFP1, FFP2, and FFP3. Before reading the following information, please check the category to which the respirator belongs - this is indicated on the packaging and on the respirator.

**APPLICATIONS:** • FFP1: protection against non-toxic solid and liquid aerosols (e.g. oil-mists) in concentrations up to 4.5 x MAC/OEUTLV or 4 x APF. • FFP2: protection against non-toxic and low-to-average toxicity solid and liquid aerosols (e.g. oil-mists) in concentrations up to 12 x MAC/OEUTLV or 10 x APF. • FFP3: protection against non-toxic, low-to-average toxicity and high toxicity solid and liquid aerosols (e.g. oil-mists) in concentrations up to 50 x MAC/OEUTLV or 20 x APF.

**INSTRUCTIONS FOR PUTTING THE RESPIRATOR ON:** 1. Hold the respirator in your hand with your fingertips in the direction to the nose part. Straps:

They serve for attaching the respirator to your head. Leave it hang loose along your hand.

2. Attach the respirator firmly to your face with its nose part onto your nose bridge.

3. Stretch the lower strap, pull it over your head and put it under your ears. Stretch the upper strap, pull it over your head and put it above your ears. •Only for the respirators with adjustable straps over the head: adjust the tension by pulling the loop on each of the straps when putting the respirator on (the tension can be adjusted by sliding out the back side of the buckle).

4. Slightly press the nose part of the respirator and shape it around your nose.

5. (a) Testing putting the respirator without the exhale valve on: Cover the respirator with both your hands and breathe out heavily. (b) Testing putting the respirator with the exhale valve on: Cover the respirator with both your hands and breathe in heavily. You should feel a negative pressure inside the respirator. If you feel air flowing around your nose, tighten the nose part. If the air penetrates along the edges of the respirator, adjust the position of the straps.

6. If you feel any difficulty in breathing, if the respirator is damaged or bent or if it is not possible to adjust it properly on the face, immediately replace the respirator.

Careful observing of the following instructions is necessary for safe use of the respirator.

### WARNINGS AND LIMITATIONS:

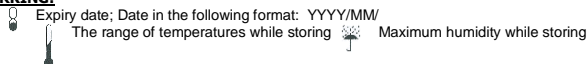
Always make sure that the product is:

- suitable for the given application;
- properly applied/put on;
- worn for the entire period of exposure;
- replaced when necessary.

• It is necessary to select the most suitable protective equipment, instruct its user, observe the instructions for its use and store the product in a suitable method to achieve optimum protection of the user against certain contaminants in the air. Failure to observe these instructions or unprofessional handling of the respiratory protection device during the user's stay in a contaminated environment can have negative impact on the user's health and lead to a serious disease, permanent disablement or death.

1. Before you start to use the respirator, consult it with a hygienist or safety officer, whether it meets your needs. 2. This product does not supply oxygen. The product shall be used only in adequately ventilated areas containing sufficient oxygen allowing life support. Do not use the respirator in areas with oxygen concentration under 19 %. 3. Do not use it even if the concentration of pollutants immediately endangers health or life. 4. Do not use it in explosive environment. 5. Immediately leave the working place and seek out fresh air, if (a) breathing becomes difficult, or (b) you fill vertigo or dizziness. 6. The protective effect of the respirator could be reduced by a beard or hair on the face or certain significant features of the face. 7. Never modify or adjust the respirator. 8. For single use only. No maintenance is required. Dispose of the respirator after use. 9. Keep the respirator in its original box protected against direct sun when storing, transporting and before its use. Store in accordance with the instructions specified on the packaging.

### MARKING:

Dispose of in accordance with valid legal regulations.

The packaging is not suitable for contact with food.

**NOTE:** MAC = Maximum Admissible Concentration. OEL = Occupational Exposure Limit. TLV = Threshold Limit Value. APF = Assigned Protection Factor.

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